




SURGYNEEDLE

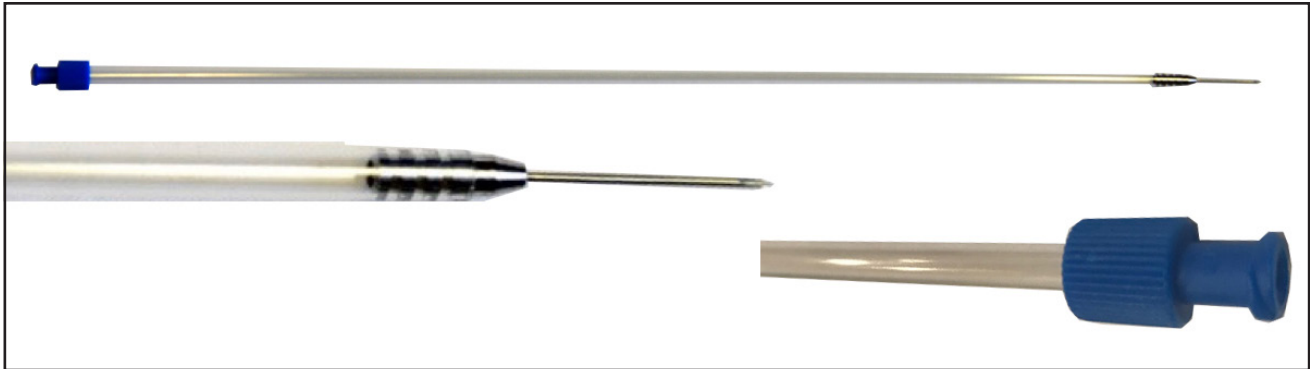

The US Surgitech introduces SurgyNeedle, A Patent Pending Clear & Flexible 5mm Disposable Aspiration Needle. It's designed with a working length of 40cm for the simple, safe aspiration and injection of fluids from cysts and organs.

FEATURES AND BENEFITS INCLUDE:

- Sterile disposable single use eliminates maintenance, cleaning and sterilization
- 5mm / 40cm long with Luer-Lock connection for safe, simple insertion through a Trocar
- Sharp beveled needle tip for controlled cyst / organ penetration
- Flexible shaft allows for more maneuverability
- Clear and transparent provides greater fluid visualization

Product	Code	Description	Pack Qty
SurgyNeedle	780-411	5mm Aspiration Needle 15 Gauge Needle	10 / Box
SurgyNeedle	780-412	5mm Aspiration Needle 21 Gauge Needle	10 / Box

FOR ORDERING
PLEASE CONTACT
Sales@surgyneedle.com



DIRECTIONS FOR USE

SURGYNEEDLE 5MM ASPIRATION NEEDLE 15-GAUGE NEEDLE Item # 780-411
21-GAUGE NEEDLE Item# 780-412

The Following information should be read before using this device.

INDICATIONS:

U.S. Surgitech's SurgyNeedle 5mm Clear Flexible Aspiration Needles have applications in minimally invasive abdominal procedures for the aspiration of fluids from cysts and organs.

CONTRAINDICATIONS:

The instrument is not intended for use when minimally invasive techniques are contraindicated.

DEVICE DESCRIPTION:

U.S. Surgitech's SurgyNeedle 5mm Clear Flexible Aspiration Needles have a beveled stainless-steel needle tip for the controlled penetration of cysts and organs. They have a clear, transparent 5mm shaft and a proximal luer-lok connection for the attachment of a standard syringe for fluid aspiration.

DIRECTIONS FOR USE:

- Using sterile technique, remove the instrument from the packaging. To avoid damage, do not flip the instrument into the sterile field.
- Insert the SurgyNeedle into the abdomen through a 5mm trocar.
- Using appropriate technique, insert the needle point of the U.S. Surgitech 5mm Aspiration Needle into the cyst/organ to be drained.
- Attach an appropriately sized syringe to the proximal luer-lok connector and aspirate the desired volume of fluid by withdrawing the syringe plunger.
- Retention of the aspirated fluids will allow cytological testing, if required.

PRECAUTIONS/WARNINGS:

- These devices are designed for single use only and are not to be re-used or re-sterilized. Do not use if packaged is opened or damaged
- Endoscopic procedures should be performed only by physicians with adequate training and knowledge of these procedures. In addition, medical literature should be consulted regarding techniques, hazards, contraindications and complications prior to the performance of these procedures.
- Before endoscopic instruments and accessories from different manufacturers are utilized together in a procedure, verify compatibility.

SINGLE USE PRECAUTION:

These devices are designed and sold for single use only. Re-processing and, or, re-sterilization is not permitted. The effects of any unauthorized reprocessing or re-sterilization can result in the following complications:

1. Cross contamination due to ineffective reprocessing/re-sterilization
2. Mechanical fatigue, and associated failure, due to the effects of the reprocessing/re-sterilization method.

The U.S. Surgitech 5mm Aspiration Needles are supplied sterile for single patient use only. Discard after use.