

# SurgyCut Disposable Laparoscopic Scissor

## Instructions for Use



### INTENDED USE:

The SurgyCut fully disposable scissor is designed to cut, dissect, manipulate, and/or cauterize various tissues during endoscopy/laparoscopy, general, vascular, gynecological, and thoracic surgical procedures.

### FEATURES:

- a. RF Post
- b. Shaft Rotation Cone



### INSTRUCTIONS FOR USE:

1. Using sterile technique, remove the scissor from the packaging.
2. Inspect the scissor shaft and tip prior to use for any damages.
3. Connect the monopolar cable to the RF post and ensure that the RF post is fully covered before use.

### PRECAUTIONS & WARNINGS:

- This device is designed for single use only and are not to be reused, reprocessed, or re-sterilized. Do not use if the package is opened or damaged.
- All procedures should be performed only by physicians with adequate training and knowledge of these procedures. In addition, medical literature should be consulted regarding techniques, hazards, contraindications and complications prior to the performance of these procedures.
- Before any instruments and accessories from different manufacturers are utilized together in a procedure, verify compatibility.

### SINGLE USE PRECAUTION:

This device is designed and sold for single use only. Reprocessing and/or re-sterilization is not permitted. The effects of any unauthorized reprocessing and/or re-sterilization can result in the following complications:

- Cross contamination due to ineffective reprocessing/re-sterilization
- Mechanical fatigue and associated failure due to the effects of reprocessing/re-sterilization

**NOTE: SurgyCut is supplied sterile for single patient use only. Discard after use.**